CATTANI GROUP

FAST & STERIL 3 FAST & STERIL 3 IMPRONTE

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FAST & STERIL 3 IMPRESSIONS

Cold sterilizer packed.

 **magnolia**
Chemical Professionals

FAST & STERIL 3 e FAST & STERIL 3 IMPRONTE

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FAST & STERIL 3 and FAST & STERIL 3 IMPRESSIONS

It is a cold sterilizing agent in single use packs made up of three bags containing the ingredients. Only in this way, having the active principles packed separately and sealed, you can be certain of using a safe product at any time until the expiry date.

The pre-measured amount eliminates the possibility of dilution errors.

FAST & STERIL 3 and FAST & STERIL 3 IMPRESSIONS

The product generates a "nascent" peracetic acid in a calibrated and controlled amount.

FAST & STERIL 3 and FAST & STERIL 3 IMPRESSIONS

Thanks to the appropriate amount of Active Principle, it guarantees the sterilization of all surgical instruments and dental impressions in alginate, polyether, silicone and rubber without corrosion or dimensional changes.

It is effective against viruses, fungi and bacteria, including sporogenous ones, in short times.

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HOW TO USE:

Tear the three bags with Component A, Comp. B and Comp. C open along the pre-cut area and pour the content into the decontamination tank containing 2.5 L of water for Fast&Steril 3 or 1 L of water for Fast&Steril 3 Impressions.

Stir to dissolve the components.

Wait for 10'.

Immerse the instruments / impressions completely.

After 15', remove the instruments / impressions and rinse them (with sterile water if you wish to keep sterility).

After 36 hours from preparation, the solution is no longer suitable for instrument sterilization and can be used to treat the spittoon unit or disposed of directly into the water drainage system.

WARNINGS:

Use protective gloves.

Squirts or splashes may discolor fabrics.

Excessive immersion times (more than 60') may damage instruments.

Impressions in alginate or polyether have to be removed immediately after 15' of disinfection.

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CHARACTERISTICS AND ADVANTAGES:

• SINGLE-DOSE:

Practical, it ensures the correct dosing to avoid measurement errors in the preparation of solutions. Separation of powders with different specific gravity is also avoided.

• ACTIVE INGREDIENTS PACKED IN THREE BAGS:

The Active Principles generating peracetic acid are all easily soluble in water, including cold, without leaving undissolved granules that may obstruct holes or damage hinges. Packing in three separate bags is a guarantee of duration because it prevents the interaction and spoilage of ingredients.

• EFFECTIVE FOR 36 HOURS:

after activation **Fast&Steril 3 / Fast&Steril 3 impressions** is effective for 36 hours, therefore it can be used for 2 days consecutively. This allows you to halve the costs of use, in comparison with other peracetic acid products which are only effective for max. 24 hours.

• BIODEGRADABLE:

it is not necessary to entrust the product's disposal to authorized companies, since it can be disposed of through the sewage with minimal effect on the environment and without costs.

• 25 SINGLE-DOSE PHIAL PACK:

Each pack contains 25 single-dose phials to obtain 2.5 L **Fast&Steril 3** solution or 1 L **Fast&Steril 3 Impressions** solution.

• MULTIPURPOSE:

Fast&Steril 3 is designed to be used with instruments in 2.5 L of solution.

Fast&Steril 3 impressions is designed to treat impressions in 1 L of solution.

Both can be used also for the suction system.

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Magnolia is part of the Cattani group which is a leader in the production of dental aspiration systems. Nobody better than the manufacturer can suggest suitable disinfectants for his products.

Via Natta, 6/A

43122 PARMA - ITALIA

Tel. +39 521 607.604

Fax +39 521 399.967

www.cattani.it/magnolia

e-mail: info.magnolia@cattani.it