





## THE FIRST CHAIR WITH INTEGRATED IONIZER FOR ROOM SANITIZATION

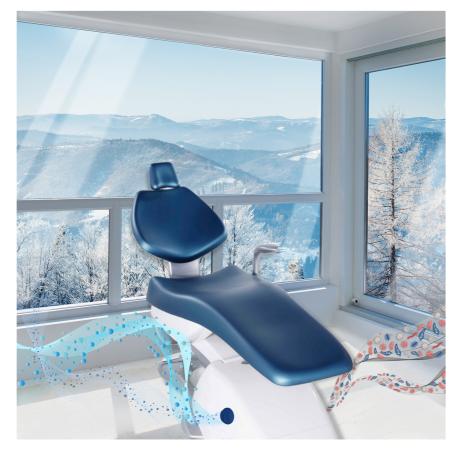



## Integrated ionizing device for continuous room sanitization

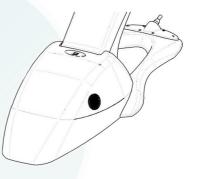

- Air exchange rate guaranteed by the entire STING ION chair system (ionizer and forced ventilation): *29 m3/h*.
- Continuous room sanitizing power until the device is switched off.
- Removes up to 99% of bacteria, viruses, VOCs and odors.
- Powerful deodorizing action that gives the environment an intense scent of fresh air.
- Natural process that promotes the controlled formation of specific ionic particles in the air through a purposely generated electric field.
- Does not use or produce chemicals and does not pollute the environment.
- Easy maintenance:
  - o Every 1.000 hours of operation (5 months approximately) it is necessary to perform a minor cleaning of the ionizing tube and the outer grid on the glass tube.
  - o Every 10.000 hours of operation (4 years approximately), or in any case when the glass becomes dull or a whitish oxide coating appears on the perforated metal foil inside the glass tube, it is necessary to replace the entire ionizing tube.



Tecnodent S.r.l. Via 63° Brigata Bolero, 24 – 40033 Casalecchio di Reno (BO), Italy Tel. +39 051 6131143 – info@tecnodent.com – www.tecnodent.com